

Restylane®  
redefining beauty™



Immediate and long-lasting  
wrinkle correction<sup>1</sup>



[www.RestylaneUSA.com](http://www.RestylaneUSA.com)



The answer to long-lasting wrinkle correction is *Restylane*.<sup>®1</sup>

Before choosing a wrinkle correction treatment, there are a few things you should know.

**Q What is Restylane®?**

**A** *Restylane* is a safe and natural cosmetic dermal filler that restores volume and fullness to the skin to correct moderate to severe facial wrinkles and folds, such as nasolabial folds.

**Q What is Restylane made of?**

**A** *Restylane* is the first cosmetic dermal filler made of non-animal-based hyaluronic acid. Hyaluronic acid is a natural substance that already exists in the human body and is used to provide volume and fullness to the skin. The hyaluronic acid in *Restylane* is a crystal-clear gel called NASHA™ or Non-Animal Stabilized Hyaluronic Acid. NASHA is biodegradable and fully biocompatible with human hyaluronic acid.

**Q How is Restylane different than other hyaluronic acid-based products?**

**A** Unlike rooster-derived hyaluronic acid-based products, *Restylane* is free from animal proteins. This quality limits any risk of animal-based disease transmission or allergic reaction. *Restylane* contains trace amounts of gram-positive bacterial proteins and should not be used by patients with a history of allergies to such material. Additionally, *Restylane* has the highest concentration of hyaluronic acid available in a dermal filler.

**Q Is Restylane® safe?**

**A** Yes. *Restylane* has been used by dermatologists and plastic surgeons since 1996 worldwide. There is limited risk of animal-based disease transmission or allergic reaction. *Restylane* is fully biocompatible and allergy testing is not required. *Restylane* contains trace amounts of gram-positive bacterial proteins and should not be used by patients with a history of allergies to such material. After your treatment, you might have some redness or swelling. This will normally last less than seven days. Sunbathing and cold outdoor activities should be avoided until any redness or swelling disappear. If you are pregnant, breastfeeding, or under 18, you shouldn't use *Restylane*.

**Q Can Restylane be used anywhere on the face?**

**A** *Restylane* has been used in more than 1.5 million treatments in over 60 countries to correct a variety of wrinkles. In the U.S. the FDA has approved *Restylane* for wrinkles and folds like the ones from the nose to the corners of the mouth. Your doctor will consult with you prior to your injection regarding common treatment areas.

**Q How long does Restylane last?**

**A** *Restylane* is proven to deliver long-lasting results. Studies have shown that *Restylane* effects generally last for about six months! *Restylane's* stabilization technology helps maintain the cosmetic effect until the injected material is nearly gone.



**Nasolabial folds before treatment**



**After treatment with three  
Restylane® 0.7 mL syringes**

Individual results may vary.

**Restylane®** is indicated for the treatment of moderate to severe facial wrinkles and folds




**Nasolabial folds before treatment**




**After treatment with two  
*Restylane*® 1 mL syringes**

Individual results may vary.

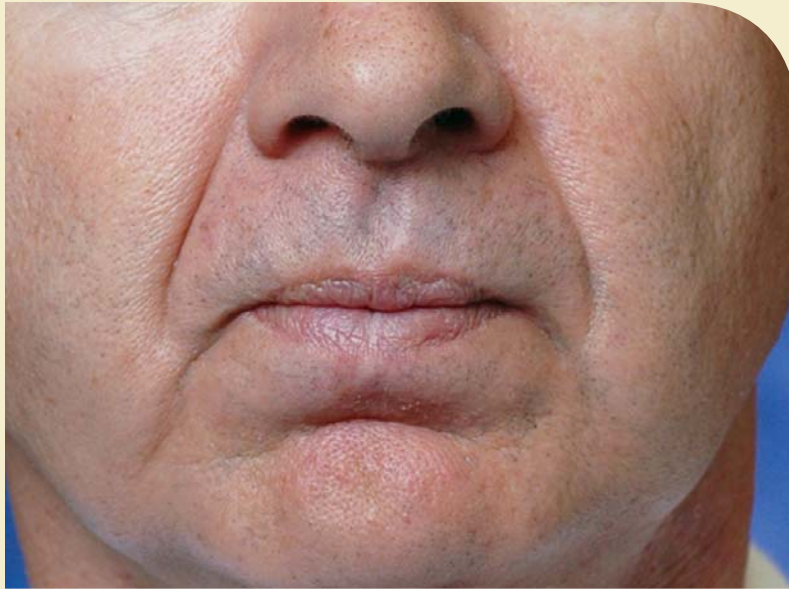

**Nasolabial folds before treatment**

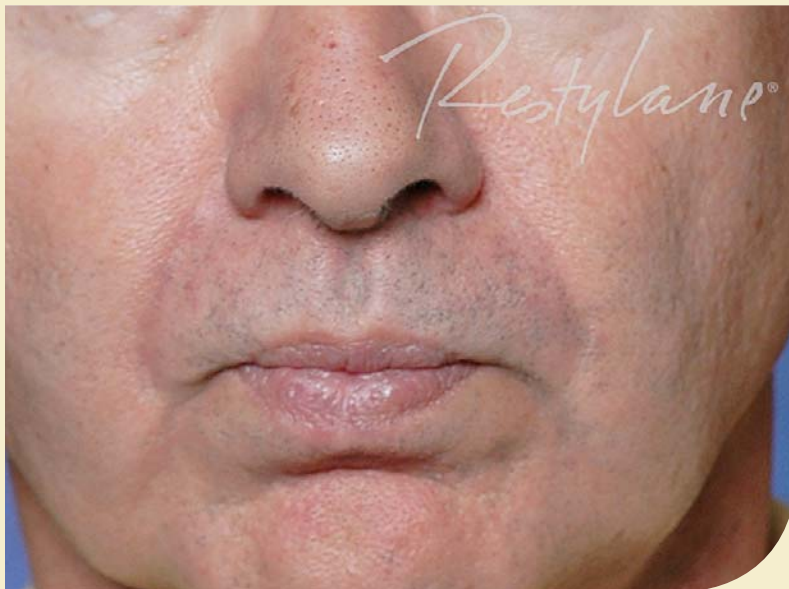

**After treatment with three  
*Restylane*® 1 mL syringes**

Individual results may vary.



**Nasolabial folds before treatment**



**After treatment with two  
*Restylane*® 1 mL syringes**

Individual results may vary.

***Restylane*® maintains its shape using  
the body's own moisture.  
Water holds the key to the long-lasting  
results you can expect.**



**Q How often should I have *Restylane*® treatments done?**

**A** Treatments are typically scheduled twice a year. Having a touch-up treatment before the product has fully dissipated may enhance the lasting effect.

**Q What should patients do prior to treatment?**

**A** *Restylane* requires no pretesting, but you should take a few precautions before being treated. Avoid using aspirin, non-steroidal anti-inflammatory medications, St. John's Wort, or high doses of Vitamin E supplements prior to treatment, because these may increase bruising or bleeding at the injection site. Also, if you have previously suffered from facial cold sores, discuss this with your physician. He/She may consider prescribing a medication to minimize recurrence.

**Q How is *Restylane* different from Botox® Cosmetic?**

**A** *Restylane* is a natural, cosmetic filler. In contrast, Botox Cosmetic is a drug that blocks nerve transmission and relaxes the underlying muscles. Many patients have been very pleased after receiving treatment with both products.

**Q How is *Restylane* different from bovine collagen?**

**A** Bovine collagen is derived from animals (cows) and requires an allergy test. *Restylane* can be administered immediately—no waiting is required.

**Q Do the injections hurt?**

**A** *Restylane*<sup>®</sup> is injected directly into the skin in tiny amounts by an ultrafine needle, resulting in minimal discomfort. The procedure is simple and convenient, and results are practically instantaneous. To optimize your comfort during the short procedure, your physician may anesthetize the treatment area.

**Q How much do *Restylane* treatments cost?**

**A** *Restylane* is a customized procedure based on your specific needs, so the cost will vary from patient to patient. In general, the cost of *Restylane* is comparable to the cost of similar procedures. Because *Restylane* is long lasting,<sup>1</sup> it proves to be very economical over the long term. Please consult your physician to determine the best recommendation for your particular needs.

**Q Who is Medicis Aesthetics™?**

**A** Medicis Aesthetics is a subsidiary of Medicis,<sup>®</sup> a well-established dermatology company dedicated to helping patients attain a healthy and youthful appearance and self-image.

### Safety information

After your treatment, you might have some redness or swelling. This will normally last less than seven days. Sunbathing and cold outdoor activities should be avoided until any redness or swelling disappear. If you are pregnant, breastfeeding, or under 18, you shouldn't use *Restylane*. For complete product and safety information, visit [www.RestylaneUSA.com](http://www.RestylaneUSA.com)



**Beauty** is now far less fleeting than previously thought.

## Why wait?

If you want to enjoy a smoother, more youthful appearance for about six months<sup>1</sup>, ask your physician if *Restylane*<sup>®</sup> is right for you.

- *Restylane* is a safe, natural formulation
- A simple convenient procedure
- Over 1.5 million treatments worldwide

## Register now!

Go to [www.RestylaneUSA.com](http://www.RestylaneUSA.com) to find out how you can join *Restylane Rewards*<sup>™</sup> and earn gifts for being a loyal customer. After each treatment, you'll be rewarded with your choice of gift cards from Barnes & Noble or Nordstrom, or a prepaid *Restylane* Visa Gift card<sup>2</sup>.

## Don't miss out!

*Restylane*<sup>®</sup>  
redefining beauty<sup>™</sup>

*Restylane*<sup>®</sup> is available only through a licensed practitioner.

### Reference

1. Narins R S, Brandt F, Leyden J, Lorenc ZP, Rubin M, and Smith S. A randomized, double-blind, multicenter comparison of the efficacy and tolerability of Restylane versus Zyplast for the correction of nasolabial folds. *Dermatol Surg* 2003; 29:588-595.
2. Limitations and restrictions apply, visit [www.RestylaneUSA.com](http://www.RestylaneUSA.com) for details.

*Restylane* and *Restylane* (stylized) are registered trademarks of HA North American Sales AB. All other marks (or brands) and names are the property of their respective owners.

